CUDA[®]

SURGICAL

CUDA[®] ELITE Headlight System Operation and Service Manual

Model SYC-410FS (PAL)
Model SYC-400FS (NTSC)

ST Technologies
6018 Bowdendale Avenue
Jacksonville, FL 32216 USA

Customer Service: 904 208 2291
Toll Free 877 814 2237

RMS UK, Ltd.
28 Trinity Road
Nailsea, Somerset BS48 4NU
United Kingdom
TEL: 01275 858891

TABLE OF CONTENTS

1. INDICATION FOR USE
2. GENERAL WARNINGS
3. ASSEMBLY
FIGURE 1
4. CAMERA CONTROL AND FUNCTIONS
5. OPERATION
6. MAINTENANCE
7. CLEANING
8. REPLACEMENT PARTS
9. WARRANTY AND REPAIR

1. INDICATION FOR USE

Congratulations on the purchase of your new Headlamp Camera system.

This headlamp camera system is designed to provide a crisp picture and deliver illumination from a high intensity light source to the surgical site.

This Operator Manual will help you to install the device and optimally integrate it with other components of your system. It will also instruct you how to make adjustments and how to keep it clean. It will give you maintenance and service guidelines as well as recommendations for best performance results.

2. GENERAL WARNINGS

The user of this product should be thoroughly familiar with the use and care of this product.

The user should carefully study the manual before making any attempt to use this product clinically. Follow the instruction in the operating manual of other manufacturer's equipment when used with this product.

Before each procedure, carefully inspect the fiber optic cable to insure the proper maintenance and sterilization (when recommended).

Light sources using high intensity lamps will produce heat as well as light. The high output of such systems can cause burns.

Use care not to point the light directly at the eyes during operation; the brilliant light can cause severe eye discomfort.

FIRE HAZARD: DO NOT DRAPE OR COVER THE LIGHT SOURCE OR THE FIBER OPTIC CABLE WHILE IT IS OPERATING. DO NOT PLACE THE FIBER OPTIC CABLE END ON THE DRAPES WHILE IT IS OPERATING.

Caution

The metal end fittings of the fiber optic cable get hot during operation. Allow to cool outside the light source for at least five minutes before touching.

3. ASSEMBLING THE HEADLIGHT CAMERA SYSTEM

Before continuing, remove the protective caps.

To attach the fiber optic cable, insert the instrument end into the headlamp module until it snaps in place (figure 1). A bifurcated cable fits into the cable clips on both sides and rear of the headband.

To remove the cable, grasp the body of the module in one hand and the metal part of the cable in the other and pull up. Do not pull the cable by its legs.

Remember, the fiber optic cable is made of glass fibers. When adjusting the cable on the headband, do not bend the cable at an angle of more than 60 degrees.

Next, place the headband on your head; you can make size adjustments by turning the knob on the rear of the headband until it feels comfortable. For more comfort you can adjust the cross band.

On the sweatband, the adjustments are made by the rear and the cross member. Open the sweatband at the rear and adjust until comfort is reached; press unit together with the Velcro etched.

Connect the camera cable to the CCU at the terminal marked “camera,” make sure not to use excessive force.

Once the headlamp module and cables are connected, the unit is ready for use. Adjustment to the module can be made by grasping the module, (take care not to touch the lens), and move it in the required position. Use a sterile cloth to avoid contamination. If the picture is not centered on the monitor, the camera module can be adjusted by tilting it forward or backward to center the picture. If the image is not in focus, thread the lens by hand clockwise until it stops turning. Point camera at a target 18” away and thread lens until image is in focus. The system comes with gown clips to relieve the weight of the cable from the headband. One gown clip is attached below the shoulder; the other is used to connect the cable to the surgeon’s gown. This eliminates the risk of tripping or dropping the illuminator end.

The metal portion of the fiber optic cable (at the lightsource end) gets hot during operation. Allow cooling for a minimum of five minutes outside the illuminator before touching.

FIGURE 1


4. CAMERA CONTROLS AND FUNCTIONS

Information

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can generate radio frequency and, if not installed and used in accordance with the instrument manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Camera control unit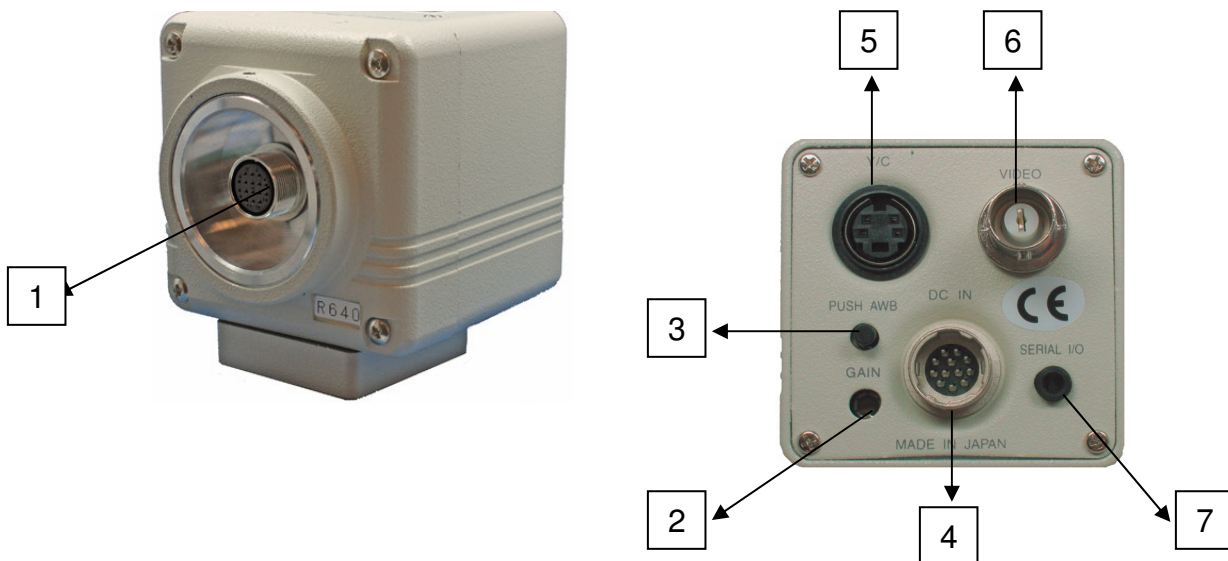

No.	Name	Function
1	Camera terminal	Connects to the camera head.
2	Gain adjust	Turning this screw will increase or decrease the gain.
3	AWB switch	Sets the white balance.
4	DC in 12V terminal	Accepts the DC power supply (12V)
5	S-VIDEO terminal	Connects terminal to S input of a monitor or a VCR, or recording system.
6	BNC VIDEO terminal	Connects terminal to the video input terminal of a monitor or a VCR.
7	Remote terminal	Controls the functions via a serial port.

5. OPERATION

Attach power cord to camera adapter. The camera system is ready to go. When required, the user can alter the picture quality by utilizing the following steps.

5.1 Gain Control

When more sensitivity is required you can adjust the gain by turning the gain control screw while observing the picture. Increasing the gain will also increase the noise. We recommend you increase the lighting to obtain a good picture.

5.2 White Balance

A white balance adjustment is necessary to obtain a picture with the correct color tone. This camera allows you to select the white balance adjustment. Point camera at a white object and press the AWB button on the back of the camera this will set the white balance.

6. MAINTENANCE

Storing the headlamp system in a safe place will prolong the life of the unit.

Fiber optic cables are an optical instrument.

The following guidelines will help in sustaining the headlamp systems useful life:

- Forming the cable in sharp angles and kinks should be avoided whenever possible. The internal fibers are made of glass, which will break under these conditions.
- Keep the optical faces of the cable and camera lens from touching hard surfaces, which may cause scratches on the surface. Scratches on the surface will diminish the light output.
- When cuts or punctures are visible in the silicone tubing, the cable is no longer safe to use, and should be taken out of service immediately.

7. CLEANING

The headlamp module can be wiped down with alcohol. The headlamp module lens and the camera window should be cleaned only with lens tissue, available in any camera store, by following the directions on the package.

Follow all applicable bloodborne pathogen procedures as required by OSHA and or your hospital, when cleaning, disinfecting and sterilizing.

The sweatband may be cleaned using a standard wash cycle in warm water and detergent.

The headband can be cleaned with an alcohol wipe down or washed with a luke warm solution of water and mild detergent. Rinse thoroughly.

Sterilizing of lensed fiber optic cables should be avoided, the lens at the end of the cable is not sealed and moisture will enter between the lens and the cable.

WARNING: Do not autoclave the headband, lensed cables or sweatband.

8. REPLACEMENT PARTS

Item	Part number
Fiber Optic cable, Bifurcated with lens	SYC3500B-BLUE-10
FiberOptic cable, Single with lens.....	SYC3500S-BLUE-10
Carrying Case	SYC0063
Garment Clips (Quantity 5)	SYC0062
Comfort Liner, Set	SYC-0040

9. LIMITED WARRANTY

Your fiberoptic product(s) carry a 3-year warranty from the date of shipment on workmanship and all defects of material, excluding replaceable sweatbands, headband, fiberoptic cable and lamps. Should your product prove to have such defects within three years of shipment, ST TECHNOLOGIES will repair or replace the product or component part without charge. Should your fiberoptic product(s) need servicing under this warranty, please contact ST TECHNOLOGIES for return authorization documentation. You should carefully pack the unit in a sturdy carton and ship it to the factory. Please include a note describing the defects, your name, telephone number and a return address. Warranty does not cover equipment subject to misuse, accidental damage, normal wear and tear or if transferred to a new owner without authorization from ST TECHNOLOGIES. This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

POST WARRANTY REPAIRS: Please contact your distributor or your customer support specialist for return authorization documentation.

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