CUDA®

SURGICAL

CUDA® ELITE Headlamp System Operation and Service Manual

ST Technologies
6018 Bowdendale Avenue
6025 Chester Ave.
Jacksonville, FL 32216 USA

Customer Service: 904 737 7611
Toll Free 877 677 2832

RMS UK, Ltd.
28 Trinity Road
Nailsea, Somerset BS48 4NU
United Kingdom
TEL: 01275 858891

TABLE OF CONTENTS

1. INDICATION FOR USE
2. GENERAL WARNINGS
3. ASSEMBLY
 - 3.1 Figure 1
4. LIST OF SURGICAL HEADLAMPS
5. MAINTENANCE
6. CLEANING
7. REPLACEMENT PARTS
8. WARRANTY AND REPAIR

1. INDICATION FOR USE

Congratulations on the purchase of your new CUDA ELITE Headlight system.

This fiberoptic headlamp system is designed to deliver illumination from a high intensity lightsource for surgical site illumination. This Operator Manual will help you to install the device and optimally integrate it with other components of your system. Also, it will instruct you on how to make adjustments, cleaning, maintenance and service guidelines as well as recommendations for best performance results.

2. GENERAL WARNINGS

- The user of this product should be thoroughly familiar with the use and care of this product.
- The user should carefully study the manual before making any attempt to use this product clinically.
- Follow the instruction in the operating manual of other manufacturer's equipment when used with this product.
- Before each procedure carefully inspect the fiberoptic cable to ensure the proper maintenance and sterilization (when recommended).
- Lightsources using high intensity lamps will produce heat as well as light. The high output of such systems can cause burns.
- Use care not to point the light directly at the eyes during operation; the brilliant light can cause severe eye discomfort.

FIRE HAZARD: DO NOT DRAPE OR COVER THE LIGHTSOURCE OR THE FIBEROPTIC CABLE WHILE IT IS OPERATING. DO NOT PLACE THE FIBEROPTIC CABLE END ON THE DRAPES WHILE IT IS OPERATING.

CAUTION: The metal end fittings of the fiberoptic cable get hot during operation. Allow to cool outside the lightsource for at least five minutes before touching.

3. ASSEMBLING THE HEADLIGHT SYSTEM

Before continuing remove the protective caps.

To attach the fiberoptic cable, insert the instrument end into the headlamp module until it snaps into place (Figure 1). A bifurcated cable fits into the cable clips on both sides and rear of the headband.

To remove the cable, grasp the body of the module in one hand and the metal part of the cable in the other and pull up. Do not pull the cable by its silicone tubing.

Remember, the fiberoptic cable is made of glass fibers. When adjusting the cable on the headband, do not bend the cable at an angle of more than 60 degrees.

Next, place the headband on your head with the Rear Cranial Support sitting comfortably on the back of your head; you can make size adjustments by turning the knob on the rear of the headband until it feels comfortable. Then adjust the top knob on the Sagittal strap for more comfort.

Once the headlamp module and cable are connected, the unit is ready for use. The headlight is provided with (2) prepackaged, sterile, and disposable joysticks. The joystick is to be threaded onto the headlight module as shown above and used to align and position the module.

The system comes with gown clips to relieve the weight of the cable from the headband. One gown clip is attached below the shoulder; the other is used to connect the cable to the surgeon's gown. This eliminates the risk of tripping or dropping the illuminator end.

CAUTION: *The metal portion of the fiberoptic cable at the lightsource gets hot during operation. Allow cooling for a minimum of five minutes outside the illuminator before touching.*

3.1

FIGURE 1

No.	Part Number	Description
1	SYC-0200	Cuda Elite Headband with "RCS" System
2	SYC-0040	Foam Comfort Liner Set (Set of 3)
3	SYC3500B-BLUE	3.5MM BIF. Blue Opaque Lensed Cable
4	SYC0064	Cuda 110MM Spot Module
5	SYC0067	Prepackaged, Sterile, Disposable Joystick (Set of 10)

4. LIST OF SURGICAL LAMPS

SURGICAL FIBEROPTIC HEADLIGHTS

- Comprehensive range of ergonomic, lightweight headbands designed for maximum comfort
- Headlights beam brilliant high intensity white light illumination with variable spot configuration

Each headlight includes:

- Headlight module variable spot (choice of 20-80mm or 20mm-110mm spot)
- 5.0mm bundle fiberoptic blue headlight cable for use with 20mm-80mm spot module **only** (choice of straight or bifurcated)
- 3.5mm bundle fiberoptic translucent lensed headlight cable for use with 20mm-110mm spot module **only** (choice of straight or bifurcated)
- Black duffle bag
- Two garment clips

Cuda “RCS” STYLE

New ergonomic headband with removable comfort liners front, top and rear, dual ratchet style tension adjustment (top and rear) and rear comfort support “RCS”.

<u>Part #</u>	<u>Description</u>
DRH-110B	Variable 20mm-110mm, 3.5mm translucent lensed bifurcated cable
DRH-110S	Variable 20mm-110mm, 3.5mm translucent lensed straight cable
DRH-80B	Variable 20mm-80mm, 5.0mm blue bifurcated cable
DRH-80S	Variable 20mm-80mm, 5.0mm blue straight cable

DELUXE CSC STYLE

New ergonomic headband with removable comfort liners front, top and back, clip style crown adjustment instead of ratchet style to decrease weight.

<u>Part #</u>	<u>Description</u>
DCH-110B	Variable 20mm-110mm, 3.5mm translucent lensed bifurcated cable
DCH-110S	Variable 20mm-110mm, 3.5mm translucent lensed straight cable
DCH-80B	Variable 20mm-80mm, 5.0mm blue bifurcated cable
DCH-80S	Variable 20mm-80mm, 5.0mm blue straight cable

CLASSIC STYLE

Economical Headband with rear ratchet control and clip adjustment on crown.

<u>Part #</u>	<u>Description</u>
CCH-80S	Variable 20mm-80mm, 5.0mm blue straight cable

SWEATBAND STYLE

Sweatband style headband that is made of terrycloth material and is ultra lightweight.

<u>Part #</u>	<u>Description</u>
SOH-110S	Variable 20mm-110mm, 3.5mm translucent lensed straight cable

5. MAINTENANCE

Storing the headlamp system in a safe place will prolong the life of the unit. Fiberoptic cables are an optical instrument. The following guidelines will help in sustaining the headlamp systems useful life.

- Forming the cable in sharp angles and kinks should be avoided whenever possible. The internal fibers are made of glass which will break under these conditions.
- Keep the optical faces of the cable from touching hard surfaces which may cause scratches on the surface. Scratches on the surface will diminish the light output.
- When cuts or punctures are visible in the silicone tubing, the cable is no longer safe to use; it should be taken out of service immediately.

6. CLEANING

The headlamp module can be wiped with alcohol. The lens should be cleaned only with lens tissue, available in any camera store; follow the directions on the package. Follow all applicable bloodborne pathogen procedures as required by OSHA and/or your hospital, when cleaning, disinfecting and sterilizing.

The joystick is removable and can be removed for autoclaving.

The sweatband may be cleaned using a standard wash cycle in warm water and detergent.

The headband can be cleaned with an alcohol wipe down or washed with a lukewarm solution of water and mild detergent. Rinse thoroughly.

The joystick is prepackaged, sterile, and disposable. It can be sterile soaked but DO NOT steam autoclave.

WARNING: Do not autoclave the headband, lensed cables or sweatband.

7. REPLACEMENT PARTS

<u>ITEM</u>	<u>PART #</u>
80mm spot module with mounting hardware	SYC0065
110mm spot module with mounting hardware	SYC0064
5.0mm fiberoptic blue cable, straight	SYC0011-4000
5.0mm fiberoptic blue cable, bifurcated	SYC0011B-4000
3.5mm fiberoptic blue opaque lensed cable, straight	SYC3500S-BLUE
3.5mm fiberoptic blue opaque lensed cable, bifurcated	SYC3500B-BLUE
Classic CCH style headband	SYC0010
Sweat pad, front, back, top (for use with Classic CCH headband)	SYC0054
Sweatband style headband (quantity 3)	SYC0056
Deluxe CSC style headband	SYC0057
Cuda RCS style headband	SYC-0200
CSC style comfort liner, front, back, top (for use with Deluxe CSC style headband)	SYC0059
RSC style comfort liner, front, back, top (for use with Deluxe RSC style headband)	SYC0061
Foam comfort liner (Set of 3)	SYC-0040
Joysticks (quantity 10)	SYC0067
5.0mm garment clips (quantity 5)	SYC0016
3.5mm garment clips (quantity 5)	SYC0062
Locker size carrying case	SYC0063
Duffle bag (black)	SYC0025

8. LIMITED WARRANTY

Your fiberoptic product(s) carry a 3-year warranty from the date of shipment on workmanship and all defects of material, excluding replaceable sweatbands, headband, fiberoptic cable and lamps. Should your product prove to have such defects within three years of the shipment, **ST TECHNOLOGIES** will repair or replace the product or component part without charge. Should your fiberoptic product(s) need servicing under this warranty, please contact **ST TECHNOLOGIES** for return authorization documentation. You should carefully pack unit in a sturdy carton and ship it to the factory. Please include a note describing the defects, your name, telephone number and a return address. Warranty does not cover equipment subject to misuse, accidental damage, normal wear and tear or if transferred to a new owner without authorization from **ST TECHNOLOGIES**. This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

POST WARRANTY REPAIRS: You may return your product(s) for repair, shipping prepaid to the factory. Your product will be inspected and an estimate of repair charges will be submitted to you for approval.

In the U.S call:	904-737-7611
Toll Free:	877-677-2832

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