

Floorstand Assembly for LED

1. Remove the base from its carton, placing the caster base with casters on the floor.
2. Remove (2) columns from the same box and remove each from its protective sleeve.
3. Locate the column with both ends tapered. Remove the expansion plug from the bottom end by first backing off the wingnut and then pulling it out of the column. Insert the end that held the expansion plug into the center hole of the caster base. Insert expansion plug into bottom hole of caster base making sure it also fits inside of column and tighten wing nut.
4. Insert the remaining column over the other end of the column already inserted into the caster base.
5. Remove from its carton the swivel hanger assembly, handle, mounting tray and required hardware. Install handle onto swivel hanger.
6. Install LED lightsource(s) onto tray with hardware provided. If only one LED lightsource use the holes located in the center of the tray. For dual lightsources use mounting holes as shown on diagram.
7. Insert the swivel hanger assembly into top column.
8. Install tray with LED lightsource(s) onto swivel hanger using the captive screws as shown on diagram to secure.

