CUDA[®]

SURGICAL

Headlamp System Operation and Service Manual

ST Technologies[®]
6018 Bowdendale Avenue
Jacksonville, FL 32216 USA

Customer Service: 904 208 2290
Toll Free 877 814 2237

RMS UK, Ltd.
28 Trinity Road
Nailsea, Somerset BS48 4NU
United Kingdom
TEL: 01275 858891

LIT051CUDA
(English)

Rev. E
Date of Revision: 12/23/13
Page 1 of 50

TABLE OF CONTENTS

1. INDICATION FOR USE
2. GENERAL WARNINGS
3. ASSEMBLY
 - 3.1 Figure 1
4. LIST OF SURGICAL HEADLAMPS
5. MAINTENANCE
6. CLEANING
7. REPLACEMENT PARTS
8. WARRANTY AND REPAIR

1. INDICATION FOR USE

Congratulations on the purchase of your new Headlamp system.

This fiber optic headlamp system is designed to deliver illumination from a high intensity light source for surgical site illumination. This Operator Manual will help you to install the device and optimally integrate it with other components of your system. It will also instruct you how to make adjustments, cleaning, maintenance and service guidelines as well as recommendations for best performance results.

2. GENERAL WARNINGS

- The user of this product should be thoroughly familiar with the use and care of this product.
- The user should carefully study the manual before making any attempt to use this product clinically.
- Follow the instruction in the operating manual of other manufacturer's equipment when used with this product.
- Before each procedure carefully inspect the fiber optic cable to insure the proper maintenance and sterilization (when recommended).
- Light sources using high intensity lamps will produce heat as well as light. The high output of such systems can cause burns.
- Use care not to point the light directly at the eyes during operation; the brilliant light can cause severe eye discomfort.

FIRE HAZARD: DO NOT DRAPE OR COVER THE LIGHT SOURCE OR THE FIBER OPTIC CABLE WHILE IT IS OPERATING. DO NOT PLACE THE FIBER OPTIC CABLE END ON THE DRAPES WHILE IT IS OPERATING.

CAUTION: The metal end fittings of the fiber optic cable get hot during operation. Allow to cool outside the light source for at least 5 minutes before touching.

3. ASSEMBLING THE HEADLIGHT SYSTEM

Before continuing remove the protective caps.

To attach the fiber optic cable, insert the instrument end into the headlamp module until it snaps in place (Figure 1). A bifurcated cable fits into the cable clips on both sides and rear of the headband.

To remove the cable, grasp the body of the module in one hand and the metal part of the cable in the other and pull up. Do not pull the cable by its legs.

Remember, the fiber optic cable is made of glass fibers. When adjusting the cable on the headband, do not bend the cable at an angle of more than 60 degrees.

Next, place the headband on your head; make size adjustments turning the knob on the rear of the headband until it feels comfortable. For more comfort, adjust the cross band.

On the sweatband the adjustments are made by the rear and the cross member. Open the sweatband at the rear and adjust until comfort is reached; press unit together with the Velcro etched.


Once the headlamp module and cable are connected, the unit is ready for use. Adjustment to the module can be made by means of the joystick. The joystick is removable for autoclaving.

The system comes with gown clips to relieve the weight of the cable from the headband. One gown clip is attached below the shoulder; the other is used to connect the cable to the surgeon's gown. This eliminates the risk of tripping or dropping the illuminator end.

CAUTION: *The metal portion of the fiber optic cable at the lightsource gets hot during operation. Allow cooling for a minimum of 5 minutes outside the illuminator before touching.*

3.1

FIGURE 1

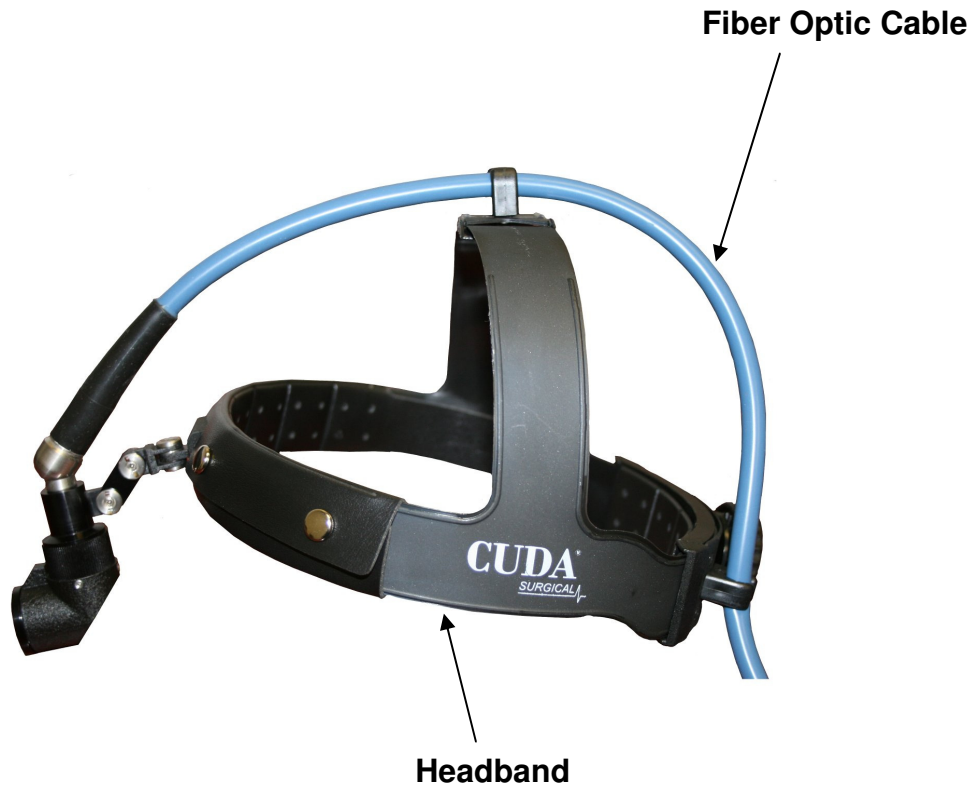4. LIST OF SURGICAL LAMPS

SURGICAL FIBER OPTIC HEADLIGHTS

- Comprehensive range of ergonomic, lightweight headbands designed for maximum comfort
- Headlights beam brilliant high intensity white light illumination with choice of fixed or variable spot configuration.

Each headlight includes:

- Headlight module (choice of fixed or variable spot)
- 5.0mm bundle fiber optic blue headlight cable (choice of single or bifurcated)
- Duffle bag
 - Two garment clips
 - Variable focus with 20mm to 80mm spot module or fixed spot with 100mm spot.

DELUXE PLUS

New ergonomic headband with removable comfort liners front & back, dual ratchet style tension adjustment (top and rear).

| <u>Part #</u> | <u>Description</u> |
|---------------|----------------------|
| DHCK5000VB | Variable, Bifurcated |
| DHCK5000V | Variable, Single |
| DHCK5000FB | Fixed, Bifurcated |
| DHCK5000F | Fixed, Single |

DELUXE

New ergonomic headband with removable comfort liners front and back, clip style crown adjustment instead of ratchet style to decrease weight.

| <u>Part #</u> | <u>Description</u> |
|----------------------|---------------------------|
| DHC5000VB | Variable, Bifurcated |
| DHC5000V | Variable, Single |
| DHC5000FB | Fixed, Bifurcated |
| DHC5000F | Fixed, Single |

STANDARD

Economical Headband with rear ratchet control and clip adjustment on crown.

| <u>Part #</u> | <u>Description</u> |
|----------------------|---------------------------|
| HLC5000VB | Variable, Bifurcated |
| HLC5000V | Variable, Single |
| HLC5000FB | Fixed, Bifurcated |
| HLC5000F | Fixed, Single |

ULTRACOMFORT

Sweatband style headband that includes breath-o-prene fabric that wicks sweat and ultra lightweight.

| <u>Part #</u> | <u>Description</u> |
|----------------------|---------------------------|
| UHC5000VB | Variable, Bifurcated |
| UHC5000V | Variable, Single |
| UHC5000FB | Fixed, Bifurcated |
| UHC5000F | Fixed, Single |

5. MAINTENANCE

Storing the headlamp system in a safe place will prolong the life of the unit. Fiber optic cables are an optical instrument. The following guidelines will help in sustaining the headlamp systems useful life:

- Forming the cable in sharp angles and kinks should be avoided whenever possible. The internal fibers are made of glass which will break under these conditions.
- Keep the optical faces of the cable from touching hard surfaces which may cause scratches on the surface. Scratches on the surface will diminish the light output.
- When cuts or punctures are visible in the silicone tubing, the cable is no longer safe to use; it should be taken out of service immediately.

6. CLEANING

The headlamp module can be wiped with alcohol. The lens should be cleaned only with lens tissue, available in any camera store; follow the directions on the package. Follow all applicable bloodborne pathogen procedures as required by OSHA and/or your hospital, when cleaning, disinfecting and sterilizing.

The sweatband and soft “sport” headband may be cleaned using a standard wash cycle in warm water and detergent.

The headband can be cleaned with an alcohol wipe down or washed with a lukewarm solution of water and mild detergent. Rinse thoroughly.

WARNING: Do not autoclave the headband, lensed cables or sweatband.

7. REPLACEMENT PARTS

| <u>ITEM</u> | <u>PART #</u> |
|--|---------------|
| Fiber Optic cable, Single | SYC0011-4000 |
| Fiber Optic cable, Bifurcated | SYC0011B-4000 |
| Standard Headband, Bifurcated | SYC0010B |
| Standard Headband, Single | SYC0010 |
| Sweat pad, front, back, top (for use with Standard headband) | SYC0054 |
| Ultra Comfort Headband, Bifurcated | SYC0037 |
| Ultra Comfort Headband, Single | SYC0036 |
| Deluxe Headband, Bifurcated | SYC0039 |
| Deluxe Headband, Single | SYC0038 |
| Deluxe Plus Headband, Bifurcated | SYC0045 |
| Deluxe Plus Headband, Single | SYC0044 |
| Comfort Liner, front, back, top (for use with Deluxe headband, straight and bifurcated) | SYC0053 |
| Joysticks (Quantity 3) | SYC0012 |
| Garment Clips (Quantity 3) | SYC0016 |
| Carrying Case (Sturdy, deluxe briefcase style) | SYC0017 |
| Duffle Bag (Nylon) | SYC0025 |

8. LIMITED WARRANTY

Your fiberoptic product(s) carry a 3-year warranty from the date of shipment on workmanship and all defects of material, excluding replaceable sweatbands, headband, fiberoptic cable and lamps. Should your product prove to have such defects within three years of the shipment, ST TECHNOLOGIES® will repair or replace the product or component part without charge. Should your fiberoptic product(s) need servicing under this warranty, please contact ST TECHNOLOGIES® for return authorization documentation. You should carefully pack unit in a sturdy carton and ship it to the factory. Please include a note describing the defects, your name, telephone number and a return address. Warranty does not cover equipment subject to misuse, accidental damage, normal wear and tear or if transferred to a new owner without authorization from ST TECHNOLOGIES®. This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

POST WARRANTY REPAIRS: You may return your product(s) for repair, shipping prepaid to the factory, with \$10.00 to cover handling. Your product will be inspected and an estimate of repair charges will be submitted to you for approval. Payment must be received before repairs are completed.

| | |
|------------------|--------------|
| In the U.S call: | 877-677-2832 |
| FAX number: | 904-733-4832 |

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