CUDA[®]

SURGICAL

ELITE 9000-II LED Headlight System Operation Manual LLS-9000- II

ST Technologies[®]
6018 Bowdendale Avenue
Jacksonville, FL 32216 USA

Customer Service: 904 208 2290
Toll Free 877 814 2237

RMS UK, Ltd.
28 Trinity Road
Nailsea, Somerset BS48 4NU
United Kingdom
TEL: 01275 858891

TABLE OF CONTENTS

- 1. INTENDED USE**
- 2. GENERAL WARNINGS**
- 3. ASSEMBLY**
- 4. MAINTENANCE**
- 5. CLEANING**
- 6. DESKTOP SMART CHARGER AND CALIBRATOR**
- 7. REPLACEMENT PARTS**
- 8. USER SERVICE**
- 9. WARRANTY AND REPAIR**
- 10. END OF PRODUCT LIFE**
- 11. CHART OF MEDICAL DEVICE SYMBOLS USED**

1. INTENDED USE

This LEDELITE 9000-II Headlight system is designed to deliver illumination from a high intensity LED for surgical site illumination.

Congratulations on the purchase of your new LED Headlight system.

The LED Headlight ELITE 9000-II is a battery powered portable headlight with an adjustable spot and uses encrypted battery technology that requires the use of approved batteries obtained from the manufacturer or authorized distributors.

Additionally, a low battery alarm is offered in this system and will beep when power is low. Decreasing the output may extend battery life and therefore the amount of warning time before power down. This alarm is a convenience and the user should plan for backup lighting for any application using this system; see General Warnings.

2. GENERAL WARNINGS

Note:

You must charge the batteries fully before using the headlight for the first time.

The user of this product should be thoroughly familiar with the use and care of this product.

The user should carefully study the manual before making any attempt to use this product clinically.

Before each procedure, carefully check the batteries state of charge (Fuel Gauge) to assure optimal duration for the procedure.

LED High Intensity Light sources using high intensity LEDs that will produce some heat as well as light. The high output of such LEDs can cause the headlamp to feel warm to the touch during operation, this is normal.

The battery charger and the battery chargers power supply are not to be used in the operating room theater or the patient area.

Use care not to point the light directly at the eyes during operation, the brilliant light can cause severe eye discomfort.

Users are responsible for providing backup lighting systems for your application when using this device.

3. ASSEMBLING THE HEADLIGHT SYSTEM

Before continuing remove the protective lens cap.

Insert a fully charged encrypted battery into the Battery Holster and ensure it is fully seated down on the connector at the bottom of the battery cavity.

Should a non-encrypted battery be installed, two beeps will be heard upon insertion indicating that an incorrect battery is being used and the unit will not power up. Insert an encrypted battery and the unit will power up.

To attach the power cable from the LED Module to the Battery Holster, insert the push-pull quick connector of the LED Module into the receptacle on the lead from the Battery Holster until it clicks in place as shown here. This connection will not come off when positively engaged. You can rotate the connectors to find the keyway and then press together and not damage the pins.

The On/Off switch is also the LED intensity control. Simply rotate the knob clockwise or counter-clockwise to produce the desired light output.

The headlight has an iris to allow adjustment of the beam spot diameter. Rotate the fluted iris adjustment ring on the headlight clockwise or counterclockwise for the appropriate spot diameter.

The battery holster is worn on a belt; on the rear side of the holster is a metal belt clip that slips over and secures to a belt. The power cable secures into the cable clips on the side and rear of the headband.

To remove the LED Module from the Battery Holster, press the small raised button on the quick disconnect device located between the LED Module and the Battery Holster and pull straight out without twisting.

Next, place the headband on your head; you can make size adjustments by turning the knob on the rear of the headband until it feels comfortable. For more comfort you can adjust the cross band.

Once the headlight module and cable are connected, the unit is ready for use.

The system comes with gown clips to relieve the weight of the cable from the headband. One gown clip is attached below the shoulder.

Once the battery is depleted into the last shown “block” on the power gauge, the unit may beep once to indicate low battery levels. The unit may beep twice to indicate further reductions in power before shut-down.

4. MAINTENANCE

Storing the headlight system in a safe place will prolong the life of the unit. The following guidelines will help in sustaining the headlight system useful life:

- Keep the optical faces of the LED Headlight lens from touching hard surfaces, which may cause scratches on the surface. Scratches on the surface will diminish the light output. Using the lens cap during transport will help protect the lens.

5. CLEANING

The headlight module can be wiped down with alcohol. The lens should be cleaned only with lens tissue, available in any camera store; follow the directions on the package.

Follow all applicable bloodborne pathogen procedures as required by OSHA and/or your hospital, when cleaning and disinfecting the product.

The headband can be cleaned with a alcohol wipe down or washed with a lukewarm solution of water and mild detergent. Rinse thoroughly.

Do not autoclave the LED Headlight module, headband, cable, battery or battery holster.

6. DESKTOP SMART CHARGER AND CALIBRATOR

Please refer to the separate battery and charger IFUs for detailed instructions on how to safely operate these devices.

7. REPLACEMENT PARTS

For replacement parts, additional or optional equipment :
Please call Customer Service @ 877 814 2237

8. USER SERVICE

The LED Headlight has no user or field serviceable components. It can only be serviced at the factory by factory trained technicians.

9. LIMITED WARRANTY

Your LED Headlight and battery holster carry a three year warranty from the date of shipment on workmanship and all defects of material.

Should your product prove to have such defects **ST Technologies** will repair or replace the product or component part without charge. Please contact **ST Technologies** for return authorization documentation.

You should carefully pack unit in a sturdy carton and ship it to the factory. Please include a note describing the defects, your name, telephone number and a return address. Warranty does not cover equipment subject to misuse, accidental damage, normal wear and tear or if transferred to a new owner without authorization from **ST Technologies**. This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

POST WARRANTY REPAIRS: You may return your product(s) for repair, shipping prepaid to the factory. Your product will be inspected and an estimate of repair charges will be submitted to you for approval. Payment must be received before repairs are completed.

- In the U.S call: 877 814-2237 (toll free)
- Customer Service 904 208-2290

10. END OF PRODUCT LIFE

In accordance with the European Waste from Electrical and Electronic Equipment (WEEE) directive, we encourage our customers to recycle this product whenever possible. Disposal of this unit must be performed in accordance with the applicable local environmental regulations.

In the US a list of recyclers in your area can be found at: <http://www.eiae.org/>.

Please contact customer service to issue a return authorization to return product to manufacturer at the end of product life.

11. CHART OF MEDICAL DEVICE SYMBOLS USED

	CE mark
	Manufacturer
	"Authorized Representative in the European Community."
	Caution, consult accompanying documents
	Do not dispose of in ordinary municipal waste, Recycle.

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